EL En

# MOBIAK<sup>®</sup>

**MEDICAL • GASES • FIREFIGHTING** est. 1977

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Μ.ΜΠΟΤΣΑΡΗ 96-98 Chania-Crete, Greece

+30-28210-63222 [www.mobiakcare.gr](http://www.mobiakcare.gr)

**MOBIAK**<sup>®</sup>  
MEDICAL • GASES • FIREFIGHTING est. 1977

## PURPOSE OF USE:

Specially designed for bedridden or immobile people.

## CONTENTS:

1. Inflatable basin with hose for easy water drainage.
2. Plastic tube & cord, these are referred to water bag (Optional).

Water Basin

Water Bag, optional

## CHARACTERISTICS:

1. Bathroom basin 21 cm depth, double reinforced for greater durability & comfort.

Also available with drain valve & hose for easy & quick water release.

With water bag in case of inaccessible access to water source, **optional (0806232)**

## PREPARATION INSTRUCTIONS FOR USE:

1. Inflate the basin. The valve is non-returnable and does not deflate if it remains open. To deflate it, keep the valve and the side walls pressed until it is completely drained from the air.
2. Place the basin on a flat surface.
3. Place the drain hose in a basin or sink, make sure it is lower than the basin to prevent water from flowing.


4. Open the drain valve while using the basin to drain the water.

5. Place the basin under the user's neck adjust until he or she feels comfortable.

**NOTE:**

The water bag illustrated is **optional**  
(0806232)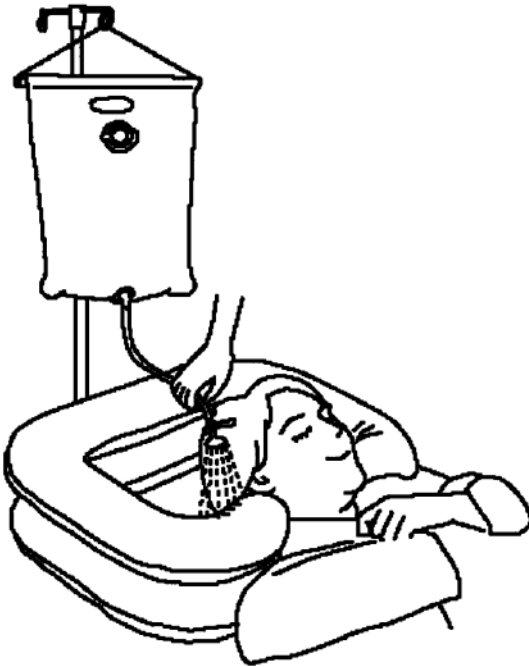

Water bag, optional

## Cleaning instructions and washing instructions

- Wash the basin with mild, antiseptic soap.
- Clean it with a wet cloth, rinse with clean water, and then dry with a dry cloth.
- Do not use detergents or other solvents.
- Do not wash it in the washing machine and do not use a dryer to dry it.
- Do not use a steam cleaner to clean it.

## EC Declaration of Conformity

We declare the conformity of the above product to the provisions of the Medical Equipment Directive 93/42/EEC and the Greek National Legislation ΔΥ8δ/Γ.Π.οικ.130648. ΦΕΚ 2198B/02-10-2009

Each product in this statement meets the technical specifications contained in its technical documentation If the appropriate maintenance will be provided and the product will be used according the “instruction of use” then the product is safe and effective and does not endanger the health safety of patient’s, user and or third parties,

This declaration is issued on the basis of a certificate of registration in the Manufacturer Register of Medical Devices of the Competent Authority (EOF).

The product belongs in Class I, Classification rule 1 & Corresponds to the Medical Device Directive: 93/42/EEC

### Protecting the environment

If your device needs replacement ,or you do not use it, consider the environment protection:

- 1) Do not dispose your appliance with other waste (this the meaning of the recycled symbol).
- 2) Contact your local authority to instruct you the recycling centers for your appliance.
- 3) Recycling your no use appliance, you protect the environment and help the exploitation of new materials from your device .

**WARRANTY:**

The product has a warranty period of **one (1) year**. The warranty covers factory defects and not damages from misuse/maintenance product. Applicable warranty period starts from the purchasing invoice date.

**Warranty Claim Form:**

**Buyers Data**

Name : .....

LOT:.....S/N:.....

Date of purchase: .....

Phone: .....

**Resellers Data**

Name : .....

Address : .....

Phone: .....

Reseller Stamp

MOBIAC S.A  
M.MPOTSARI 96-98  
Chania-Crete, Greece  
+30-28210-63222  
www.mobiakcare.gr